



Value Life

INFECTION PREVENTION & SAFETY

TOTAL SOLUTIONS

Needle-free devices
PROGRAMME GUIDE

TOTAL SOLUTIONS



Quality



Clinical outcomes



Partnership

At Vygon we pride ourselves in delivering you the best products and services, but it has now got even better.

Total Solutions delivers dedicated training and education programmes, combined with clinical tools, services and solutions which support your healthcare professionals and patients as well as your hospital.

This is all available online to ensure you get the best support easily during the most challenging times.



Healthcare professionals



Hospitals



Patients

Training programme

Supporting you with training and tools to improve clinical outcomes.



LEVEL ONE

First in line training for all clinical end users

- ✓ **Device selection based on GIRFT ensuring correct device selection**
- ✓ **Product use; best practice – ‘points to practice’ with quick reference guides**
- ✓ **Infection control and prevention – In line with ANTT ‘Scrub the hub’ training and local policies**

LEVEL TWO

Train the trainer programme

- ✓ **Train the trainer observational training**
- ✓ **Part of the Vygon e-health hub**
- ✓ **Support with audit delivery**

Working closely with our trainers to support regional collaboration and bring teams together.

Covid-19 Update

We have introduced remote training and education to ensure continued support and education is accessible. Along with tools and resources to support healthcare professionals and departments.

Conversion programme

Helping you achieve standardisation across your Trust.



STEP 1

Define

We work with you to understand your requirements, needs and timelines. To propose a solution to support Trust-wide delivery.

STEP 2

Planning

Based on your requirements and needs, we deliver a plan to support the solution with agreed timelines.

STEP 3

Implementation

We implement the plan as agreed. Delivering a conversion and training plan to support clinical and non-clinical teams, and their departments.

STEP 4

Review

During the conversion we review and sign off the delivery within each department for both product and training. Final sign off is done to complete the project.

STEP 5

Partnership

Post sign off and for the duration of your contract we support you with dedicated training and support programs. Offering both face-to-face and online delivery.

Vygon e-health portal

Introducing a healthcare portal to support both clinical and non-clinical staff with online education, training and a wealth of tools and resources.



**Clinical education
and training**



**Audits and
departmental support**



**Patient support
and education**



**Product support
and information**

Tailored tools


Providing you with customised resources and tools to support your staff department and patients.

Points to practice - bionector

Vygon ordering code: V00705 v1 06/2020

bionector® quick reference guide **VYGN**
Value Life

Points to practice

<p>Always select the appropriate connector for the cannula/catheter device. <small>Double lumen bionector® to the end of a peripheral IV cannula. Use a bionector® Octopus extension set.</small></p>	→	 <p>Single lumen bionector® Octopus extension Double lumen bionector® Octopus extension</p>		
<p>Adhere to effective hand hygiene compliance. <small>Ensure needle-free device is screened on security to end of cannula/catheter as per hospital policy and standard precaution procedures.</small></p>	→	<div style="border: 1px solid black; padding: 5px; text-align: center;"> <p>ANTT</p> <p><small>Always adopt the Aseptic Non-Touch Technique approach and adhere to hospital policy guidelines.</small></p> </div>	<p><small>Always check the needle-free device for any damage prior to dislodging it and for use.</small></p>	
<p>Ensure needle-free device is screened on security to end of cannula/catheter.</p>	→	<p><small>Scrub the hub – scrub the needle-free device port with a 70% Isopropyl alcohol and 2% CHG swab or as per hospital policy. Scrub for a minimum of 30 seconds and allow to dry for approximately 30 seconds.</small></p>	<p><small>Use a 10ml syringe with 0.9% Saline for flushing and adhere to hospital IV flushing and drug administration policy. Flush between each IV administration.</small></p>	
<p>A standardised bionector® does not require priming prior to attachment. bionector® Octopus extension sets do require priming prior to connection.</p>	→	<p><small>Use lock syringe – screw on lock into the bionector®</small></p> <p><small>Use 10ml syringe – pull in and hold (0.5-1ml)</small></p>	<p><small>Use a push-pause technique when flushing to create a turbulent flow for dislodging cannula/catheter of potential debris.</small></p>	<p><small>Remove syringe and administer medication, flushing in between each medication given (before and after) to avoid the bionector®</small></p>

bionector audit - example

Vygon ordering code: V00779 v1 06/2020

NHS
Your Organisation's Name Goes Here
Your Designator Goes Here

VYGN
Value Life



CRITICAL CARE www.vygon.co.uk

Needle-free product range

vad**site**[®]

Training programme

Level 1 & 2

Conversion programme*

Tailored tools

bionector[®]

Training programme

Level 1 & 2

Conversion programme*

Audits

Online training

Tailored tools

bionector[®]
TKO

Specialist

Training programme

Level 1 & 2

Conversion programme*

Audits

Online training

Patient education

Qimono
by VYGON

Specialist

Training programme

Conversion programme*

Audits

Online training

Patient education

Self-administration

Oncology speciality

*Assessed based on needs and complexity.

For further information, please contact: vygon@vygon.co.uk

The specifications shown in this leaflet are for information only and are not, under any circumstances, of a contractual nature.

Vygon (UK) Ltd, The Pierre Simonet Building, V Park, Gateway North,
Latham Road, Swindon, Wiltshire SN25 4DL

Tel: 01793 748800 Fax: 01793 748899 Email: vygon@vygon.co.uk

vygon.co.uk [@vygonuk](https://twitter.com/vygonuk) [in](https://www.linkedin.com/company/vygonuk) vygonuk [yt](https://www.youtube.com/channel/UCvYgONk) vygonuk [ig](https://www.instagram.com/vygonuk) vygonuk

Copyright Vygon (UK) Ltd 2020 Content correct as of: 08/2020 Code: V00772 v2



Value Life