

How to secure qimo ♂


To a fixed or a spinning Luer from an infusion line or elastomeric pump tubing.



Fixed Luer

1

Hold the fixed Luer

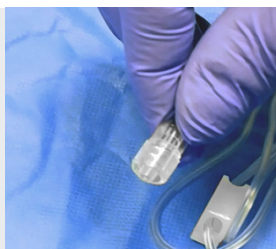


2

Introduce qimo ♂ onto the Luer



3

Maintain qimo ♂ and gently start locking the Luer onto qimo ♂

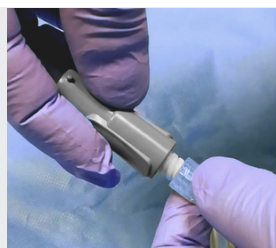


4

Hold the fixed Luer and **lock** qimo ♂ until you feel the point of resistance. Then turn it clockwise several times.



5

Turn qimo ♂ anti-clockwise to check the safe connection

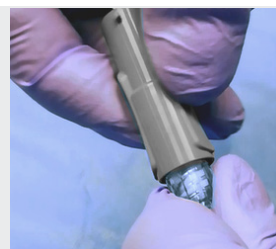

Spinning Luer

1

Push the spinning Luer to the proximal end



2

Introduce qimo ♂ onto the Luer



3

Maintain qimo ♂ and gently start locking the Luer onto qimo ♂




4

Hold the fixed Luer and **lock** qimo ♂ until you feel the point of resistance. Then turn it clockwise several times.



5

Turn qimo ♂ anti-clockwise to check the safe connection

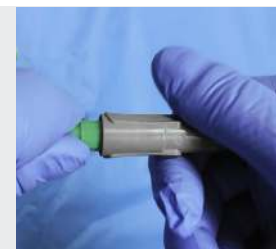

Qimono range of products include sterile medical devices class Is & IIa in compliance with Directive 93/42, appendix II & V.

Certified by G-MED notify body N°0459. Manufactured and sold by Vygon

NB: Risk of qimo ♂ disconnection from a spinning Luer is not unlikely, in particular if a lot of strength is applied on the device.