



REMOVAL STEPS

SAFETY HUBER NEEDLE COMPATIBLE WITH PRESSURE INJECTIONS


PPS® CT Single-handed removal

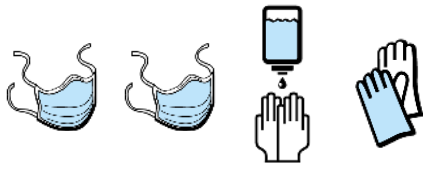


- Remove the dressing by stretching it parallel to the skin
- Remove the adhesive strips
- Remove the foam cube
- Flush the needle with saline using a push/pause flushing technique. Keep 1 or 2cc in the syringe for the positive pressure needle removal.

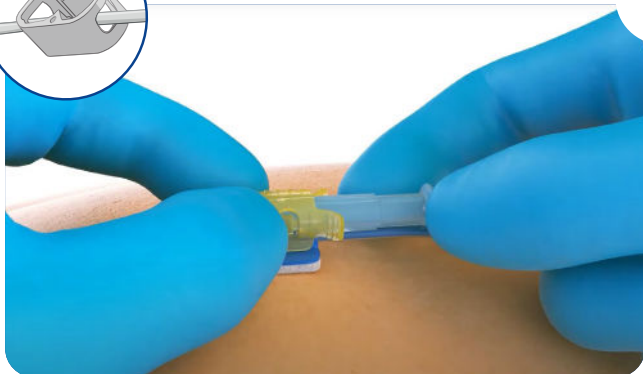

1 Hold the blue platform with 2 fingers.

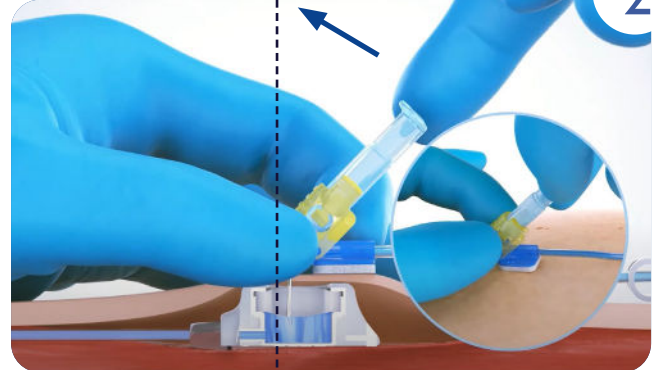


2 Lift the piston from the horizontal position to the vertical position (perpendicular to the septum) while holding the platform with 2 fingers.

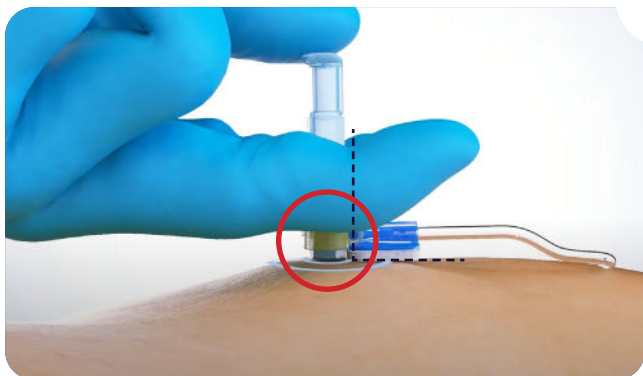


3 Make sure:

- The needle piston is perpendicular to the septum and in contact with the skin
- **To place one hand on the opposite side of the blue platform**, put your thumb on the piston, your forefinger and middle finger under the winglets (on both side of the piston).

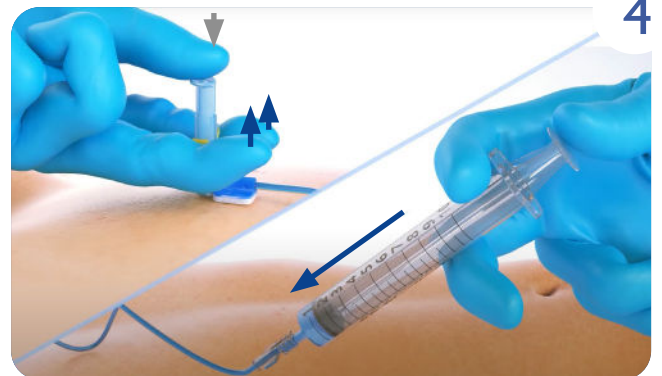

4 Maintain the piston in the vertical position with your thumb.

Then simultaneously, inject saline with the other hand and lift the extractor (with an upward movement) until there is a **CLICK** corresponding to the total locking of the needle.

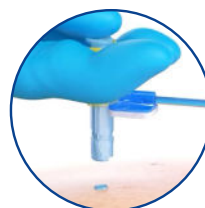

Value Life



Video available here

This final step eliminates the risk of accidental needlestick injury.

Discard the needle into the sharps safety box