

easymoov6

FAQs



EASYMOOV6 FAQs

IS IT POSSIBLE TO USE REAL FOOD OR BLENDERISED FOOD?

Yes, Easymoov6 is suitable for blended diets and thicker feeds (we recommend an IDDSI level 3 liquidised or below). In case of food prepared at home, we recommend to add extra water to dilute the feed.

CAN I USE EASYMOOV6 ON THE GO?

Yes, Easymoov6 is designed to meet all patient needs in ambulatory and stationary medical care, the Easymoov6 enteral feeding pump allows comfort and safety for patients and medical professionals in hospital, homecare and school settings.

IS EASYMOOV6 WATERPROOF?

The Easymoov6 pump cannot be immersed in any liquid. However, the pump is IP44, which allows it to be in contact with water (splashing) without any problems

DOES EASYMOOV6 MAKE A LOT OF NOISE?

No, the Easymoov6 pump features continuous rotary peristaltic motor which allows patients to feed during the night and in quieter settings.

IS IT EASY TO CONTROL?

Yes, the Easymoov6 pump allows you to change and set the flow rate, volume and nutrition mode both prior and during feeding. All the information is available on one large screen; a short learning curve allows the user to manage the pump quickly. Additionally, our specially trained sales team can provide training and helpful resources are available via our website.

WHO CAN I CONTACT REGARDING EASYMOOV6?

Our dedicated product specialists are ready to assist you with any further queries you may have. For any technical queries, please give us a call using the phone number below and ask for the Technical department



01793 748800



Capitalequipment@vygon.co.uk



www.vygon.co.uk/contact/



Value Life